AUSTRIA
 Bauerfeind Ges.m.b.H.
 Hainburger Straße 33
 1030 Wien
 P +43 (0) 800 4430-130
 F +43 (0) 800 4430-131
 E info@bauerfeind.at

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Verbandschuh
 Strap shoe
 Chaussure couvre-pansement
 Zapato para pies vendados

Rev.7 – 2019-09_107 282

All instructions for use:
www.bauerfeind.com/downloads

Material
Schaft: Polyethylen (PE)
Futter und Fußbett: Polyamid (PA) Baumwolle (CO) Polyester (PES)
Sohle: Styrol-Butadien-Kautschuk (SBR)

Material
Upper: Polyethylene (PE)
Upper lining and footbed: Polyamide (PA) Cotton (CO) Polyester (PES)
Sole: Styrene Butadiene-Rubber (SBR)

BAUERFEIND AG
 Triebeser Straße 16
 07937 Zeulenroda-Triebes
 Germany
 P +49 (0) 36628 66-35 00
 F +49 (0) 36628 66-39 99
 E info@bauerfeind.com
BAUERFEIND.COM

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en english

Dear Customer,

Thank you for choosing a product from Bauerfeind. **GloboPed Sanoped** is a high-quality medical product. Please read through these **instructions for use** carefully and if you have any questions contact your physician, medical retailer, or our Technical Service department.

Product Description

The **GloboPed Sanoped** strap shoe is a prescribable product that may only be worn under a physician's direction and only in accordance with these instructions of use and for the applications named.

Purpose

The **GloboPed Sanoped** strap shoe is a medical product for early mobilization of bandaged feet.

Indications

- For early post-operation and / or post-traumatic recovery of the foot and for wound bandages on the foot
- Treatment for foot deformities and inflammation

Contraindications and Application risks

Caution

Mobility may be compromised based on the purpose and desired function of the strap shoe. There are no known side effects on the entire body. The strap shoe should only be worn after consulting trained technical personnel¹.

There are no known hypersensitivities that cause illnesses. This device should only be worn after consulting your physician in the following clinical situations:

- Skin diseases / injuries in the body part being treated, particularly if inflammation appears. The same applies for accumulated scars with inflammation, redness, or overheating
- Acroparesthesia in the body part being treated
- Lymphoid circulation problems, unclear soft tissue inflammation around the shoe's area of application

Any open wounds on the foot must be protected by a bandage or similar device. Do not ride a bicycle or drive a car or any other vehicle while wearing the shoe. Please see your physician immediately should you notice any irregular changes in your body (i. e. increase in pain).

Application Instructions

No product liability will be assumed in the event of inappropriate use. The strap shoe should be put on while sitting and can be worn both on the left and right foot. Put on with the Velcro in snug contact to the foot, although not too tight in order to avoid pressure points on the skin respectively damages of arteries and nerves. An individual orthopedic foot insert may be worn with the **GloboPed Sanoped** strap shoe in place of the inner sole delivered with shoe. The insert may not negatively affect the inner shoe volume available to the foot. Always wear a non-skid shoe with the same sole height on your healthy foot. If necessary, a physician may prescribe that the height of the sole is to be changed by an orthopedic technician or orthopedic shoe technician.

Be sure to have your **GloboPed Sanoped** strap shoe checked regularly for functionality and closing functionality by an orthopedic technician or orthopedic shoe technician and exchanged if needed. We have tested this product within the scope of our integrated quality management system. Should you nevertheless have a complaint, please contact our local technical consultant or our customer service department. Use crutches when wearing a strap shoe. Wearing the **GloboPed Sanoped** strap shoe is only recommended in hospitals and clinics or within your own home and not in public areas. Combining the aid with other medical products (e.g. compression stockings, supports) may only be considered after having sought advice from a qualified specialist¹.

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